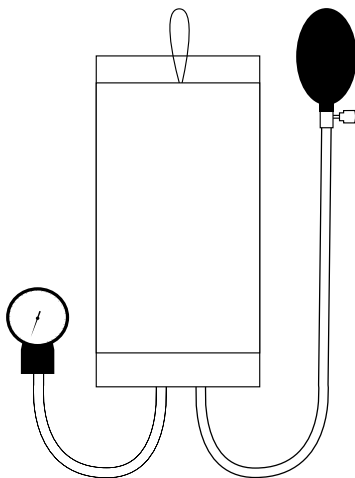
WESTMED

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Please read these instructions carefully and completely before use and observe the care instructions.

Safety instructions

- These instructions do not replace detailed instructions or training for the user.
- The product may only be used for the purpose specified in these instructions.
- The pressure infusion cuff contains small parts that could be swallowed by children and must therefore not be left unsupervised for children!

Product description

The pressure infusion cuff with manometer is a class I medical device (without measuring function). The device is explicitly not a measuring device! Although identical in construction to the MDD 93/42/EEC approved aneroid blood pressure measuring device of the same manufacturer, the manometer here only serves to provide orientation about the pressure value on the connected infusion bag. The pressure infusion cuff has been developed for practical, correct and safe use. To ensure correct use of the product, we recommend that you read the manual before use. In case of doubt, please contact Westmed Praxis.

Features

Description	Pressure infusion cuff with manometer
Size	33x17 cm
Display range	0 – 300 mmHg
Scaling	2 mmHg
Display precision	±3 mmHg (temperature: 18 °C – 35 °C) ±6 mmHg (temperature: 36 °C – 50 °C)
Air leak	< 4 mmHg/min
Use temperature	0 – 50 °C
Storage temperature	- 20 – 70 °C
Permissible ambient humidity	10 – 85 % relative humidity

Intended use

The pressure infusion cuff is a medical device that applies pressure to the inserted infusion bag used to infuse fluids. The cuff is mainly used for patients transported by stretchers, transport chairs or immobilisation systems and when no infusion pole is available or its use is not possible. The use of the pressure infusion cuff is reserved for qualified medical personnel. It is intended for use in hospitals, emergency rooms and home care.

Notes

The device should be used exclusively for the specified purposes defined in this document. Westmed declines all liability for damage caused by improper or non-intended use. The manufacturer reserves the right to make changes to the appliance and the instructions for use without prior notice in order to improve its characteristics. For correct use of the appliance, follow the instructions in this manual.

CAUTION: Never exceed the maximum pressure of 300 mmHg; the pressure of the device affects the speed of the infusion. This procedure must be carried out by qualified personnel.

Installation

Connect the tubing of the cuff with the pressure manometer and the screw valve with blower ball.

Handling

Insert the infusion bag between the mesh and the cuff, with the lower part of the bag facing the side of the tubing. It is recommended that the infusion bag is placed with the label facing upwards to make reading easier. Produce the necessary air pressure in the cuff by using the blower ball. Check that the pressure reducing valve is closed. The needle of the manometer rises and shows the respective pressure in the cuff. The necessary pressure must be determined according to the requirements of the doctor in charge. If necessary, turn up the reducing valve to adjust the pressure to the required value. To allow the air to escape quickly from the bubble, fully open the valve screw on the blower ball.

WARNING: The use of the pressure infusion cuff must only be done by qualified personnel.

Cleaning

Manometer: Use a soft and dry tissue for cleaning.

Blower ball with reducing valve: Clean with a soft, moistened tissue and let it dry well. Do not place in an autoclave.

Cuff: First remove the elastic bladder. Then wash with warm water and mild soap. Do not wash in a washing machine and do not spin dry!

Elastic bubble: Wash with warm water and mild soap. Do not let water run into it! Allow to dry at room temperature.

Maintenance

The medical devices from Westmed are CE-marked accordingly and are thoroughly checked before they are released to the market. Before each use, check that the needle of the manometer is positioned within the permissible tolerance range (± 3 mmHg, indicated by a small black line vertical to the graduations of the scale in the zero area). Manometer out of tolerance of more than ± 3 mmHg must be calibrated. For qualified technical assistance, please contact the manufacturer.

Guarantee

The device is intended for professional use only. Westmed provides the purchaser with a warranty of 12 months from the date of purchase. The manufacturer or their authorised service centres reserve the right to repair the unit, replace parts or the entire unit at their discretion. Under the terms of this warranty, Westmed is not liable for any damage that is within the scope of misuse of the device as well as if modifications/repairs are made by unauthorised personnel. There are no explicit or implied warranties of merchantability, fitness, purpose or otherwise other than those described in this manual. The life of the appliance is estimated at 2 years if the manufacturer's instructions are followed.

Art. No.: 5810000
Manufacturer No.: SL HS-20G0

0123
MDD 93/42/EEC
EN ISO 13485:2016

WESTMED distributed by
Fleischhacker GmbH & Co. KG
An der Silberkuhle 18, 58239 Schwerte, Germany
Tel.: +49 2304 9310, Fax: +49 2304 931199
info@fleischhacker.biz

Honsun (Nantong) Co., Ltd
No. 8 Tongxing Road Economic & Technological
Development Area, 226009 Nantong City,
Jiangsu Province, P. R. China

Shanghai International Holding Corp. GmbH (Europe)
Eiffestraße 80, 20537 Hamburg, Germany
SRN: DE-AR-000000001