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## Krankentragen

ultraSTRETCHER 350 / 350 R / 350 M / 710 / 710 R / 710 M / KOMPAKT

### 1 x klappbar

2310 x 550 x 140 mm (gebrauchsfertig)  
1910 x 140 x 140 mm (Packmaß)

				Gleitfüße	Rollenfüße
7,2 kg 	<b>ultraSTRETCHER 350</b>				
	REF	SAN-0101-4G	4	-	
	REF	SAN-0101-4R		-	4
	REF	SAN-0101-2G2R	2	2	
7,0 kg 	<b>ultraSTRETCHER 350 R</b>				
	REF	SAN-R0101-4G	4	-	
	REF	SAN-0101-4R		-	4
	REF	SAN-R0101-2G2R	2	2	
7,0 kg 	<b>ultraSTRETCHER 350 M</b>				
	REF	SAN-3101-4G	4	-	
	REF	SAN-3101-4R		-	4
	REF	SAN-3101-2G2R	2	2	

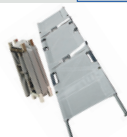
### 2 x klappbar

2310 x 550 x 160 mm (gebrauchsfertig)  
1010 x 190 x 140 mm (Packmaß)

				Gleitfüße	Rollenfüße
8,0 kg 	<b>ultraSTRETCHER 710</b>				
	REF	SAN-0104-4G	4	-	
	REF	SAN-0104-4R		-	4
	REF	SAN-0104-2G2R	2	2	
7,0 kg 	<b>ultraSTRETCHER 710 R</b>				
	REF	SAN-R0104-4G	4	-	
	REF	SAN-0104-4R		-	4
	REF	SAN-R0104-2G2R	2	2	
7,8 kg 	<b>ultraSTRETCHER 710 M</b>				
	REF	SAN-3104-4G	4	-	
	REF	SAN-3104-4R		-	4
	REF	SAN-3104-2G2R	2	2	

### 3 x klappbar

2310 x 550 x 140 mm (gebrauchsfertig)  
650 x 300 x 150 mm (Packmaß)

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**Important Information:** Before using the stretcher, please carefully read the operating manual and safety instructions. Keep the instructions for use in a safe place to avoid operating errors.

**The product's intended purpose:** The stretchers are medical products.

**Indication:** Transport of sick and injured people.

**Contraindication:** Not suitable for prolonged examination and/or immobilization during transport or spinal injuries.

**Declaration of conformity:** This product was manufactured in accordance with the current EU regulation 2017/745. A declaration of conformity for these products can be acquired from the local dealership or the manufacturer.

**Information on the safety:** The stretcher should be used only by instructed persons. The use of this type of products is part of the training program of rescue services.

- The stretcher must not be used for the transport of goods. The approved load is solely for the weight of people.
- The stretcher must not be lifted before the patient restraint straps are correctly closed.
- The stretchers may only be used with accessories or spare parts approved by the manufacturer.
- Serious incidents that have occurred in connection with the product must be reported by the user and / or patient to the manufacturer and the competent authority of the Member State in which the user and / or patient is established.

**CAUTION!** Protect the restraint straps from being walked on!

**Handling:** For reasons of space saving can stretchers be folded in length or length and width.

- When folding the stretcher in and out please be careful not to pinch the fabric between the metal parts.
- Adjustable of head- and foot-rest requires checking of the set-pins - they must fit into the setting holes securely.

**Attention!** Unfolding the hinges may squeeze in your fingers!

The stretcher is ready for use when the covering fabric is tensioned and securing pins on the hinges are arrested.

**Cleaning:** After use the stretcher is to be cleaned with water and common cleaning aids and followed sterilized. Only BGA approved aids are allowed. Ether, fuel, petrol and mineral oils may destroy the fabric cover.

**Maintenance:** In order to ensure the safety of the user and the injured party, we recommend that you have the stretcher and the straps visually inspected by an expert on an annual basis.

**CAUTION!** In the event of damage or suspected damage, remove the product from operation immediately until any occurring irregularities have been remedied.

**Repair:** Necessary repair is allowed only by the manufacturer and only OEM parts are to be used.

**Warranty:** The period of warranty is two years and starts with reception by the user. Manufacturing and material faults are covered but not damages by inappropriate handling.

**Service life:** The product service life depends on the individual operating conditions. Depending on the frequency and intensity of use of the stretcher, wear and tear is normal. Proper application is assumed here. Even with careful treatment, the material is subject to an aging process. The product should be discarded depending on its condition or after 10 years at the latest.